

Side 1


Reference Guide

Enbrel® (en-brel) (etanercept)
injection, for subcutaneous use
Single-Dose Prefilled SureClick® Autoinjector
50 mg/mL

Read all instructions in carton before use.


Questions? Call 1-888-4ENBREL (1-888-436-2735)
or visit www.injectingenbrel.com

Guide to parts

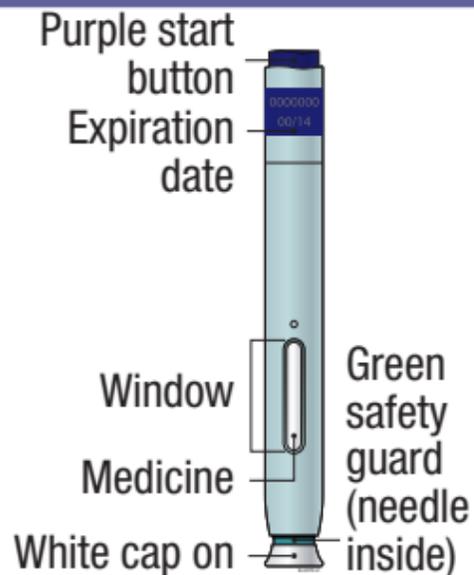


Wait at least 30 minutes
for the autoinjector to
come to room temperature
before giving your injection.

Prepare and clean your
injection site.

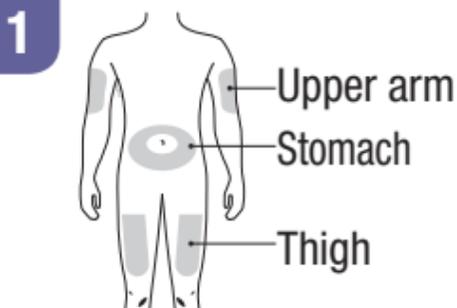

Pull the white cap straight off only when you are ready to inject.
Do not leave the white cap off for **more than 5 minutes**.
This can dry out the medicine.

2

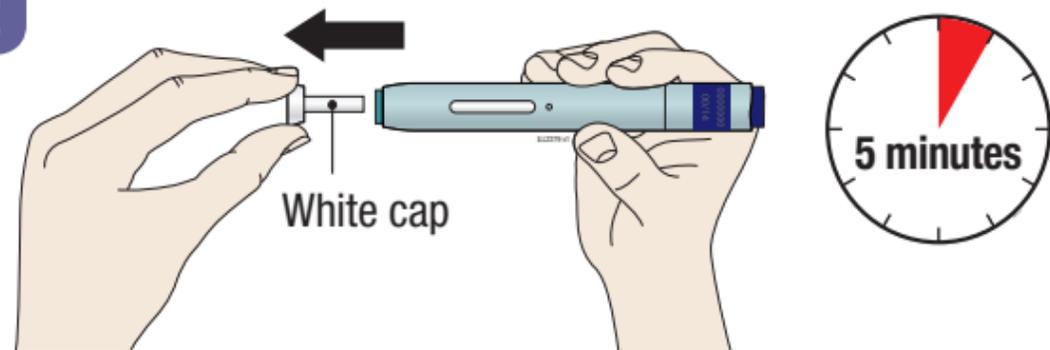

Turn over to continue

Side 2

Reference Guide

Enbrel® (en-brel) (etanercept) injection, for subcutaneous use
Single-Dose Prefilled SureClick® Autoinjector
50 mg/mL

AMGEN®

Manufactured by:
Immunex Corporation
Thousand Oaks, CA 91320-1799
© 1998, 2019, 2021-2022 Immunex Corporation. All rights reserved.

Read other side first


USA-916-83378
34767

Revised: 8/2022 v9

♻️ This printed material is recyclable.


Stretch or pinch your injection site to create a firm surface.

3 Stretch method:

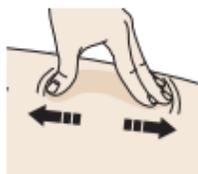

or


Pinch method:

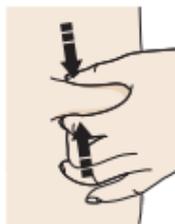

Put the green safety guard (needle inside) on your skin at 90 degrees.

4


Green safety guard (needle inside)

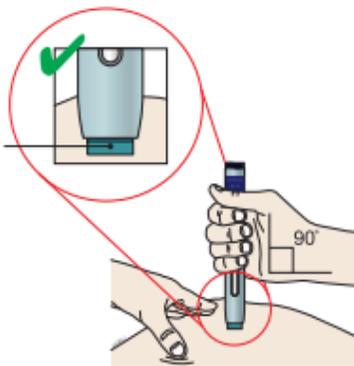

Firmly push down the autoinjector onto your skin until it stops moving.


5

When you are ready to inject, **press** the purple start button. You will hear a click.

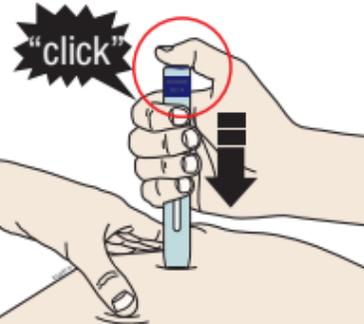

Keep pushing down on your skin for about 15 seconds.

6

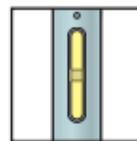

Window turns from clear to yellow when the injection is done.

